TERMOSIGILLATRICE
THERMAL SEALER

OPERATOR MANUAL

OM2001MU
Rev. 03
Date April 2021

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Dear Doctor,

First of all, thanks for the trust you put on us by choosing the NEWSEAL Thermal Sealer. Our company is at your complete disposal to provide any further information or clarification concerning this equipment.

The Thermal Sealer you purchased has been properly manufactured in compliance with international specifications.

Remind to read this manual carefully before using the Thermal Sealer, since an improper use could affect the correct operation and accelerate the wearing-out of its components.

In case of doubts on the operation do not hesitate to contact the reseller that sold the product.

Thanks for Your trust
NSK Dental Italy S.r.l.

No portion of this publication can be printed, transmitted, rewritten, stored in a data recovery system, translated in any foreign or computer language, in any form or by any devices, without a written consent by **NSK Dental Italy**.

Information in this manual is subject to change without any warning or prior notice and does not represent a commitment on the part of the vendor.

1. Familiarisation	15
2. General	17
3. Installation	18
4. Safety instructions	19
5. Operation	20
6. Troubleshooting	21
7. Maintenance	21

Purpose of this manual is to provide the operators with correct instructions for a proper use of the appliance.

If the Thermal Sealer is not correctly installed and operated, the manufacturer cannot be considered responsible for any possible breaks and malfunctions.


Please, check that the packing is whole and there no evident damages or missing parts (see packing list).

IN CASE OF DAMAGES OR MISSING PARTS, PLEASE IMMEDIATELY INFORM AND IN DETAIL THE FORWARDER, DENTAL X INC. AND ITS AREA AGENT.

1. Familiarisation

Devices used in the thermal tealer NEWSEAL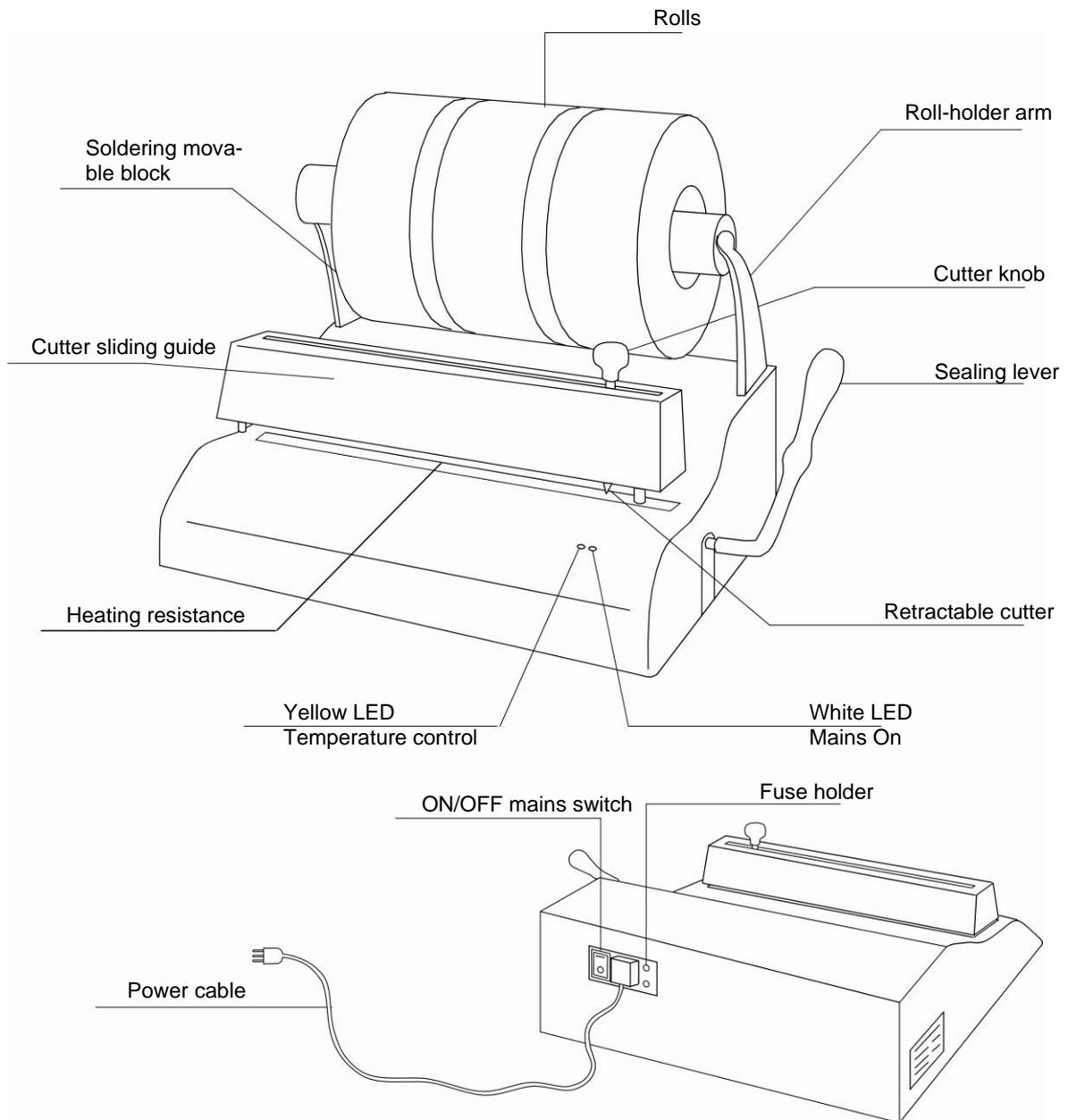


ENGLISH

Main features

- 10 mm sealing, standard-compliant
- Automatic control heating resistance
- Return of the blade after cutting
- Automatic sealing start by pulling down the lever, and with acoustic and visual indication (flashing of the yellow Led)
- Automatic sealing stop, with acoustic and visual indication
- Metal case painted with epoxy powders


Package dimensions and weight

Width	500 mm
Depth	400 mm
Height	250 mm
Gross weight	8.7 Kg

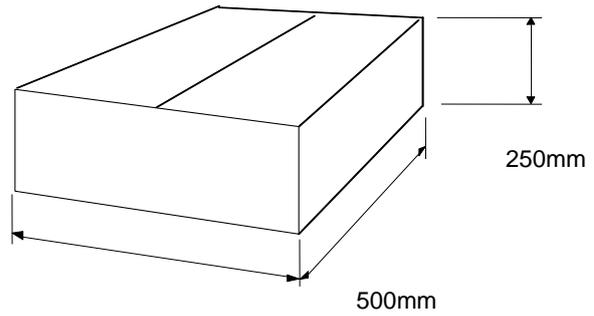
External dimensions

Width	435 mm
Depth	310 mm
Height	160 mm without rolls

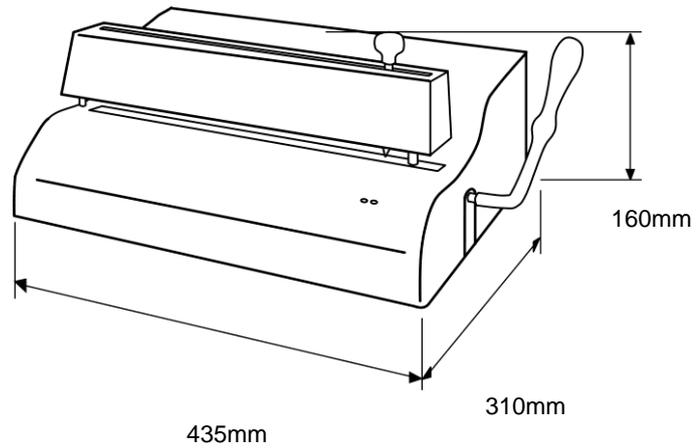Dimensions of the model complete with roll-holder and roll

Width	435 mm
Depth	310 mm
Height with roll - holder and roll	330÷430mm

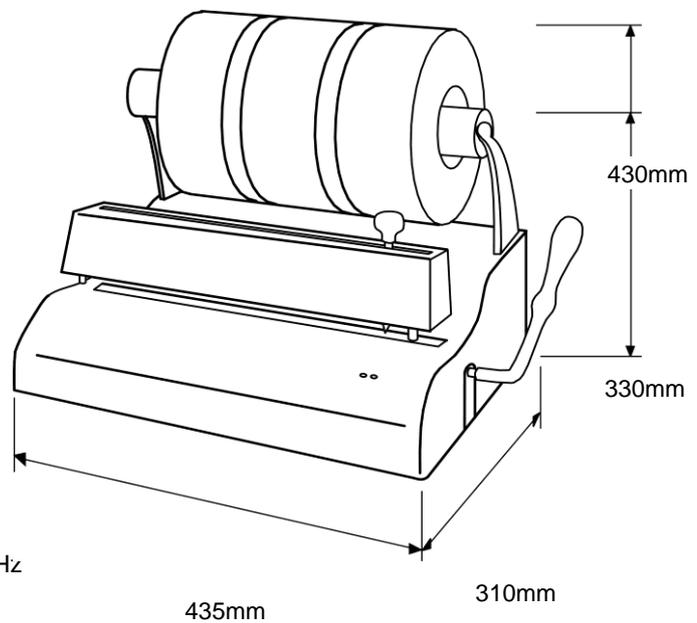Technical data

Net weight:	7 Kg
Power supply:	230 V 50/60 Hz
Max power:	400W
Fuse:	5 x 20 4A T

ENGLISH

2. General

Even when it is not strictly necessary, the instrument wrapping indicates a special care paid to the hygiene and sterilization and it is especially effective for a proper infection safety.

Wrapping the instruments means to preserve their sterile condition also during the period following the sterilization and until their use on the patient.

A properly sealed envelope makes up a protection barrier preventing any contamination from the external environment.

For this to be true, it is necessary to use envelopes that have been properly sealed.

NEWSEAL performs a uniform sealing of 10 mm width according to the international rules.

The enhanced technology guarantees efficiency and reliability for the sealing system, avoiding any risk of overheating and burning.

Inexperienced staff can easily use this equipment with a very quick training.

GENERAL RECOMMENDATIONS

In order to maintain the instruments for a long time and avoid any problems, we recommend to read the following instruction carefully.

1. Wrap the instruments one by one.
2. If more instruments must be wrapped together make sure to place instruments of same material (on the contrary, dark spots could appear at the contact point due to electrolysis).
3. We recommend to use good quality paper and propylene material.
4. For the sterilization of wrapped instruments, use longer sterilization times. The vapour must actually filter within the envelopes and this requires longer times.
5. The envelopes should be placed inside the autoclave with the paper surface up. This allows the vapour exit easier during the drying phase.
6. Do not overload the sterilization chamber! Leave a free space for the steam circulation.


3. Installation

PRELIMINARY INFORMATION

- NEWSEAL thermal sealer has been tested at the factory.
- On receiving the sealer, please control that the packing is whole and keep it in case it should be shipped.
- Open the packing and control that the delivery corresponds exactly to the technical specifications and that no apparent damage is present.
- In case of damage or missing parts inform the forwarder, NSK Dental Italy S.r.l. and its area agent immediately.

TO INSTALL THE EQUIPMENT FOLLOW THESE INSTRUCTIONS

1. Place the sealer on a flat surface and away from water sources.
2. Install the sealer in a ventilated environment and away from heat sources.
3. In case of model provided with roll-holder, insert the supports of the roll-holder into their slots, then fit in the roll on the supports.
4. Insert the power plug, and the sealer will be ready.

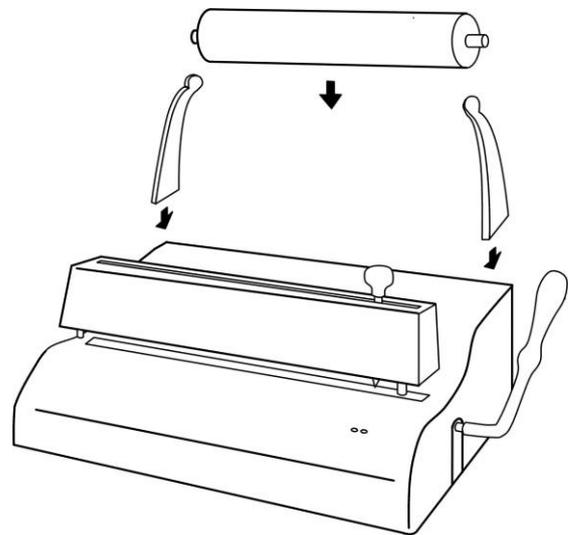WARNING

The first sealing operations could produce a slight smoke and an unpleasant smell. This fact, due to substances used on assembling the machine, is temporary and will disappear after a few sealing operations.

4. Safety instructions

Before using Your **NEWSEAL** Thermal Sealer, read these safety instructions carefully. Non-compliance with these standards could cause incidents or damage to the appliance.

- Do not introduce fingers in the sealing area when the machine is on.
- Do not introduce fingers under the protection cover of the sealing area when the lever is lowered; a proper protection grid avoids any possible contact with the heating resistance.
- Do not insert your fingers in the sliding surface of the cutter.
- Keep the area around the sealer clean and dry.
- Never disassemble the thermal sealer when it is powered, since the high internal voltage could be dangerous.
- In case of impossibility for disconnecting the power, turn to OFF the breaker of the electrical switchboard and place a **Work in Progress** sign on the breaker if this is far and not visible by the person that is carrying out the maintenance.
- Clean the appliance with a soft damp cloth after unplugging the power supply cable. Do not use solvents on the product label.
- Check that the machine is electrically grounded in compliance with the standard of the country of installation.
- Do not remove the rear label and check that the sealer is supplied with proper voltage value.


ELECTRICAL CONNECTION

Check that the power rating indicated on the label on the rear side of the sealer corresponds to the values available in the place of installation.

The equipment must be connected through the mains cable to a socket provided with a ground connection in compliance with the standard of the country of installation.

CAUTION: Do not bend the cable and do not place any object on it.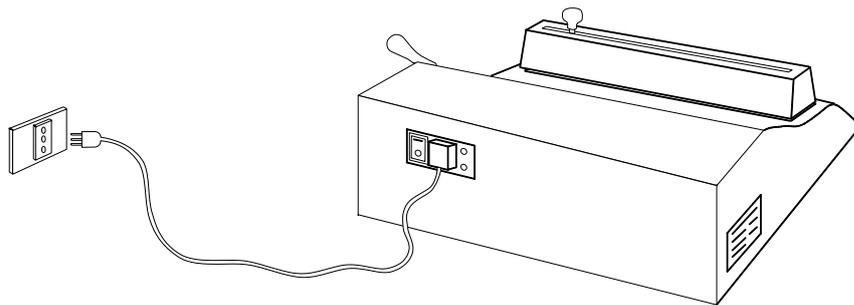

5. Operation

1. Connect the power supply cable of **NEWSEAL** Thermal Sealer to a mains socket after having checked the electrical requirements reported on the label.
2. Turn on the equipment by the power switch placed on the rear side; the white light indicator will turn on. The waiting time for the ready status of the heating resistance (turn on of the yellow lamp) changes from 1,5 and 2 minutes.
3. Place a sterilization roll on the special stand. In case of model with a roll-holder, put the rolls in the rod then on the supports.
4. Insert the envelope under the roll behind the heating element and push it until it goes beyond the heating element and appears on the front side.

CAUTION:

the envelope must have the transparent (polypropylene) side up.

5. Advance the envelope for all the length of the instrument to be enveloped.
6. To seal the envelope grasp the lever and pull it forwards and down. In this way the soldering device is lowered pressing the envelope against the heating element..
7. On complete lowering of the lever the yellow indication will start to flash to signal that the sealing is in progress (this phase lasts for about 6 seconds).
8. When sealing is over the yellow light switches on steady and an acoustic message is played.
9. Cut the envelope before releasing the lever. To do this push the knob placed over the sealing element and move it firmly along the sliding track of the cutter both rightwards and leftwards.
10. Return the lever to rest position in order to lift the sealing element and free the envelope just cut..


WARNING:

If switched on and unused for more of 25 minutes, the sealer will go off automatically (yellow lamp will switch off too). For switching on again the sealer, lower the sealing lever or operate off/on the mains switch.


11. In order to seal the opposite side of the envelope and so close the instrument within its protective cover, the envelope must now be turned and inserted under the soldering device from the front; then carry out a new sealing.

WARNING:


Avoid the sealing operations along the outer margins of the heating element; place the sterilization roll on its special stand, so that the envelope may slide along the middle part of the heating element.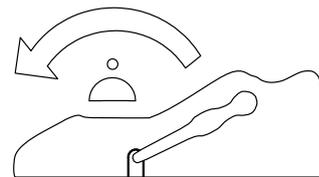

Cut the envelope.

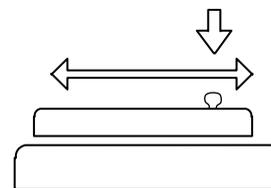Return the lever to rest position.

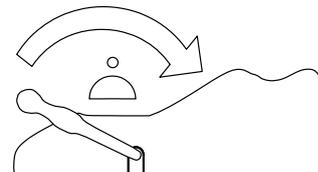6. Troubleshooting

Problem	Solution
The power white light stays off.	Check the electrical plug Check the fuses
The protection sheath on the resistor is burnished.	Call the service for the replacement
The paper is not clearly cut.	Replace the cutter or call the service
The device operates but the yellow light is not flashing.	Call the service for the LED replacement
The heating element smokes or sends out a bad smell.	If the problem persists after the first few sealing operations, call the service

The sealer is provided with a special self-controlling heating resistance and with resettable safety thermostat.

If the fuse is burnt, call the service for the replacement or send the appliance directly to the nearest service center.

We mark that the function of the fuse is for protecting the thermal sealer from possible worse and more expensive damage and that the fuse is a proper element required by the safety standards.

USE ONLY AND EXCLUSIVELY ORIGINAL SPARE PARTS AND APPLY TO AN AUTHORIZED TECHNICAL SERVICE CENTER.

NSK Dental Italy is not responsible for any faults or damage caused by appliances tampered with by non authorized personnel or in case of fault of parts that are not original.

7. Maintenance

To clean the outside of the appliance, use a damp cloth after making sure that the power supply cable has been unplugged.

For an extra maintenance call the Service.

NSK Dental Italy S.r.l.
Via dell'Agricoltura, 21
36016 Thiene (VI) - Italy
Tel. +39 0445 820070

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We reserve the right to
make any alterations
which may be due to
technical improvements.

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