

## Avila II: Set-up instruction

1. Take Avila II from carry case



2. Release the legs by turning the main lever and lift the legs (cables should be taut). Release the main post to center position; then lock the main lever.



3. Lift seat and it will automatically lock in place



4. Lift armrest by pulling T-handle; then, adjust chest and headrest height and angle by using the levers.



## **Avila II: Take-down instruction**


1. Release chest pad cam lever and lower chest pad at the lowest position



2. Using T-handle, lower armrest at the lowest position (parallel to chest pad)



3. Lower seat by holding seat with one hand while pushing seat support forward with the other hand

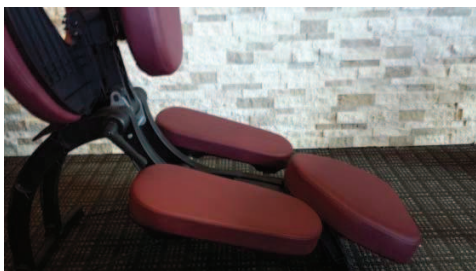

4. Loosen main lever by turning counter-clockwise and lower main post



5. Fold front legs under chair frame and tighten main lever



6. Make sure the main lever is positioned properly



7. Loosen headrest height adjustment knob and headrest cam lever



8. Avila II should be stored in carry case shown below

