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INSTRUCTIONS FOR USE OF THERMOFORM PARAFFIN PLASTIC COMPRESS

Please read these instructions in full before use. We will not be liable for any damages incurred through improper use or a failure to comply with these instructions.

Preparation:

Before every application, check the compress for any damage and make sure it is in a flawless condition. Only ever warm up the paraffin compresses in a water bath up to a maximum temperature of 70°C.

IMPORTANT: Never let them boil! After approximately 15 to 20 minutes, the paraffin will have liquefied and the compress be ready for an application of approximately 30 to 60 minutes.

Application:

Do not place the warm compresses directly onto the skin since you could burn your skin! Always use a towel or a cover. The warmth then discharged will be optimally administered and the compress will not cool off rapidly.

Areas of application:

- Circulatory disorders
- Muscle tension
- Rheumatic disease, sciatica
- Older bruising, sprains, strains
- For any therapy in which the application of warmth is indicated.

Caution is required for:

Sensory disorders, inflammation

Important:

Only warm up the compress by letting them float in a water bath!
The water must never be allowed to boil!
Never warm up the compress in the oven or the microwave!
Danger of fire if the compress is warmed up without water!
Keep the pad away from sharp or pointed items!
Store the compress in a dry area and only warm them up for use!
Never bend them when they are cold!

Cleaning:

Cleaning the compress with a mild soap solution. All commercially available, water-soluble surface disinfectants in a maximum of 2% dilution.

Disposal:

You may return them to us free of charge for recycling purposes. Small quantities of pads may be disposed of with the household waste.

Warranty for defects:

We will effect replacement free of charge in cases of defective material or manufacturing errors (liability for defects) over a 24-month period from the date of manufacture. We will not be liable for damages caused mechanically (e.g. by pins or needles) or through chemical impact (caustic disinfecting agents) during use or storage. We will not be liable for any form whatsoever of direct or indirect injury to persons and/or damage to property caused through improper use and/or a failure to comply with the instructions for use.

Variants:

THERMOFORM organ compress	24 x 32 cm	# 1020
THERMOFORM universal compress	24 x 42 cm	# 1030
THERMOFORM neck wrap	10 x 30 cm	# 1040
THERMOFORM shoulder compress	with ties	# 1050